


Clean & Collect (Catheter)

Note: Please do not use disinfectants. **Catheter with lumen** – please run standby irrigation during catheter removal from the patient. Avoid liquid penetrating the handpiece. Do not brush the device or put through automated cleaning.

PRECLEANING

1. Immediately after use wipe the catheter down with a damp, lint-free cloth.

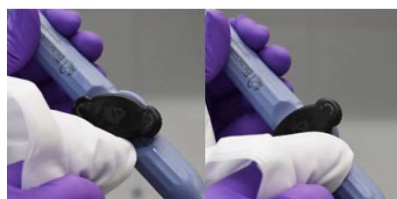

Hint: Clean the spaces in the starting position (relaxed state) and the bent position.



Hint: Clean from proximal to distal (from handle to the catheter tip).

2. Dry the catheter with a lint-free cloth from proximal to distal.



Catheters with lumen: Rinse the lumen thoroughly with clean water. Avoid any liquid penetrating the handpiece.



Use a Luer cannula and draw it up with aqua or tap water. Rinse the lumen catheter with gentle pressure.



Then inflate the Luer cannula with air and repeat the procedure. Once again, make sure that there is no visible contamination on the product.

PACKAGING

Set the catheter in the **starting position** (relaxed state). **Gently loop the catheter** and put it in the transparent bag. Please use **one bag per product**. Avoid kinks by placing the handle towards the bottom of the bag. **For new products, the label from the original device** (manufacturer information) must also be put into the bag. Place the bag into the Vanguard box.



Check out our precleaning clip:

